MINI-SYSTEM Total-Fix® /MIDI-SYSTEM Total-Fix®

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Parasite transport, preservation and concentration system with a Formalin, Mercury and PVA-Free fixative

The Mini/Midi-System is a disposable plastic device for the collection, fixation, filtration and concentration by sedimentation of intestinal Parasites from fecal specimens. The collection vial contains a formalin-free fixative that preserves the morphology, and the two built-in filters eliminate the food debris providing a clean sediment.

The concentration procedure is a modification of the Formalin Ether (Ritchie method), as recommended by WHO. The Total-Fix®(1) patented fixative does not contain formalin, alcohol and heavy metals and provides the same preservation activity as Formalin based fixatives.

The filtration is performed by two filters of 400 and 250 microns, the optimal size for obtaining a clean sediment and a good recovery of helminths eggs and protozoa. The collection vial contains glass beads for facilitating the homogenous suspension of the specimen. After fixation, a sedimentation tube is screwed to the bottom of the collection vial, forming a closed system, which is then centrifuged. The addition of Ether or Ethyl Acetate, normally needed to allow the filtration, is not required as the filtration happens during the centrifugation. After discarding the supernatant, an aliquot of the sediment is examined microscopically.

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The concentration systems are available in two different formats:

Mini-System: 150 tubes of 13mm diameter with 4ml of fixative
Midi-System: 98 tubes of 30mm diameter with 10ml of fixative

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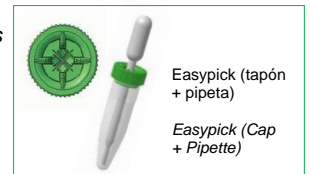
- Sedimentation cone
- Wooden sticks
- Pipettes
- Cone Cap / Easypick

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- Microscope Slides and Coverslips
- Centrifuge
- Lugol's Iodine
- Microscope

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The test kit must be stored at room temperature for the duration of the shelf-life indicated on the label. Dispose of using specific methods for the destruction of material with biological content. The content of the tube is toxic, consult the legislation for toxic substances. If this is not possible, hire an authorized company for it. The container is made of 100% recyclable polypropylen.

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1. All patient samples should be treated as infectious material and protective gloves should be used. Correct laboratory guidelines should be followed at all times.
2. Total Fix® is a hazardous solution, do not swallow. If accidental ingestion or spillage occurs, consult the safety data sheet or contact immediately the Toxicology center (contains methanol).
3. Avoid contact with skin and eyes. In case of contact flush with water. If irritation develops, contact a physician immediately.
4. To ensure optimal recovery of parasites it is recommended to take three samples from the patient on successive days.
5. Do not use the test kit beyond expiration date.

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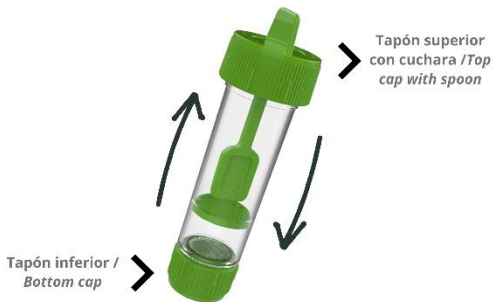
1. Avoid the use of anti-diarrheal or laxative medications before collecting the sample.
2. Collect the fecal specimen in a clean dry recipient.
3. Avoid contamination of the specimen with urine or water.
4. Transfer one spoonful of sample into the collection vial. (Do not open the bottom cap of the tube in any moment).
5. Mix to homogenize the diluted specimen.
6. Close the tube tightly, the sample is ready for processing.

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- 1.** Agitar vigorosamente el tubo conteniendo la muestra durante 30 segundos para facilitar la disgregación de la muestra.
Shake the tube containing the sample vigorously for 30 seconds to facilitate disintegration of the sample.

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If the sample is very compact, 3 drops of surfactant such as Tergitol (or similar) 20% can be added to improve homogenisation. A drop of Lugol's can be added to the slide to facilitate visibility of the structures. It is recommended to examine the whole area systematically to ensure a correct diagnosis.

- 3.** Centrifugar 3 minutos a 700 g (2000 rpm).
Centrifuge 3 minutes at 700 g (2000 rpm).

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Carefully discard the supernatant and re-suspend the pellet with 1-2 drops of water or saline. Transfer a small amount to a slide and examine under the microscope.

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Preparing a permanent staining: prepare a slide and add a small amount of sediment, spread the sediment on the slide to prepare a fine smear. Allow to dry overnight at room temperature or for several hours (from 30 minutes to 60 minutes if the slide is thicker) in an incubator at 37 ° C. Do not use heaters, these can affect the organisms present in the sample.

NOTE: Since Total-Fix® does not contain PVA (which helps the smear to stick to the slide), albumin may be added to the slide before preparing the fecal smear. However, if the smear is completely dry before staining, it is not necessary to add albumin.

Finally, proceed with the staining of choice. Trichrome staining is recommended, although iron hematoxylin may also be used

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(1) Medical Chemical Corporation (MCC)