TEST DE GRAHAM / TEST DE GRAHAM SAFE & CLEAN

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The REAL Test de Graham is the best solution for the detection of *Enterobius vermicularis*, a small intestinal parasite known as pinworm.

Laboratory diagnosis of pinworms is performed by retrieving eggs from the perianal area using the Graham's Test and subsequent microscopic identification.

Our REAL Test de Graham consists of a ready-to-use adhesive-taped slide for easy and efficient collection. Manufactured from the highest quality glass to ensure optimal visualisation.

REAL Test de Graham Safe & Clean

The REAL Test de Graham Safe & Clean is an improved version of the conventional Graham Test.

It includes our revolutionary flexible plastic cap that facilitates collection, while preventing direct contact between the patient's skin and the glass, preventing cuts and scratches, as well as decreasing the likelihood of accidental infection during sample collection.

The REAL Test de Graham Safe & Clean is packaged in a practical, durable plastic box that includes instructions for sample collection, while protecting the slides during transport.

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The test kit must be stored in a dry and cool place for the duration of the shelf-life indicated on the label. Dispose of using specific methods for the destruction of material with biological content.

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1. All patient samples should be treated as infectious material and protective gloves should be used. Correct laboratory guidelines should be followed at all times.
2. Wash hands before and after sample collection
3. Do not bring the glass slide close to the skin if it is broken or cracked.
4. To ensure optimal recovery of eggs it is recommended to take three samples from the patient on successive days.
5. Do not use the test kit beyond expiration date.

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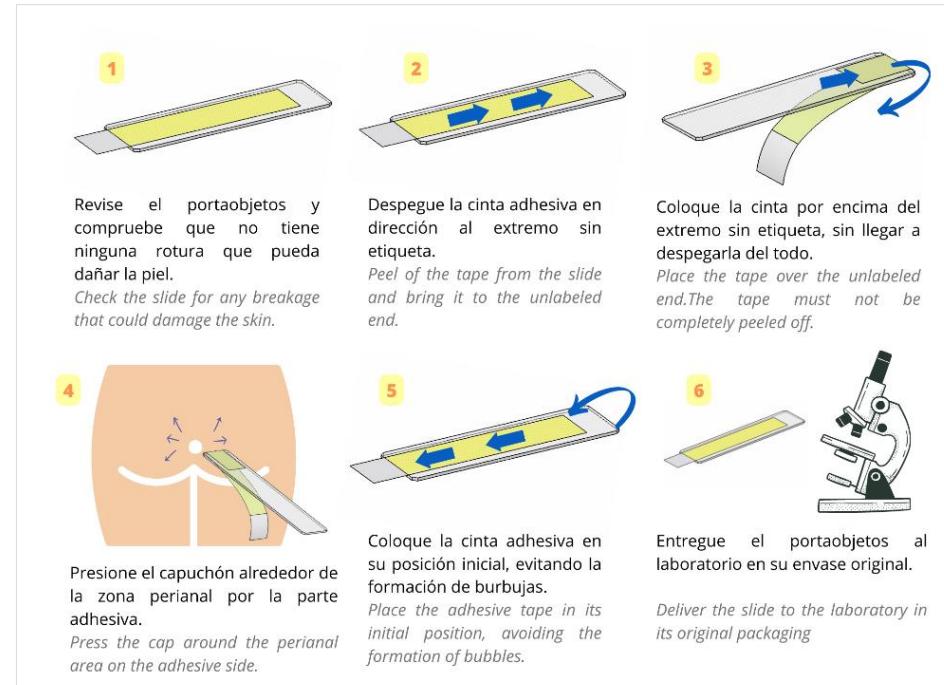
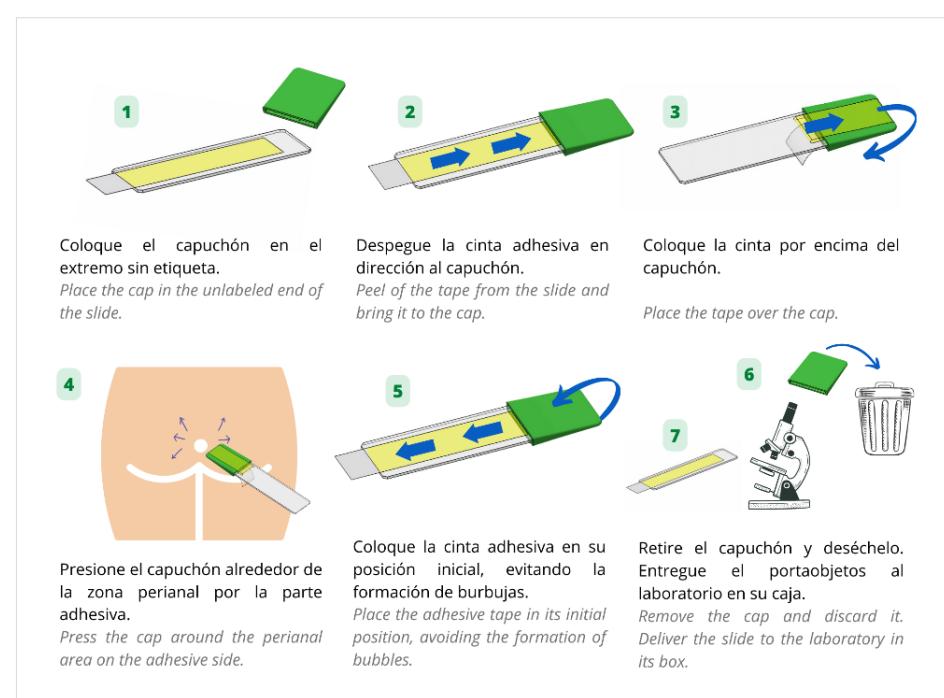
1. Collect the sample at first hour of the morning.
2. The sample should be taken prior to daily washing.
3. Handle with care everyday objects that have been in contact with the patient, especially clothing and bed sheets.
4. *E. vermicularis* eggs are highly infective, so it is recommended to perform the diagnostic test on all cohabitants.
5. Identify the packaging with the patient's name and date.

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