

REAL[®] SALIVA VIRAL

SAMPLE COLLECTION KIT SIN REACTIVO

Ref.RBMSALV01SR
Ref.RBMSALV100SR

1 unit
100 units

DESCRIPTION


REAL SALIVA Viral Sample Collection Kit Sin Reactivo provides a fast and safe procedure all-in-one for saliva sample collection and transport. This presentation doesn't contain preservative solution, the sample must be sent to the laboratory for further analysis after the collection.

Features

- Used for the collection and transport of 1 or 2 ml saliva samples.
- Painless and non-invasive sample collection.
- Allows DNA and RNA isolation with excellent quality and integrity.
- Compatible with all DNA and RNA isolation methods. **We recommend our REAL kits** for the subsequent DNA/RNA purification.
- Compatible with NGS.

PRESENTATION

REFERENCE	PRESENTATION
RBMSALV100SR	RBMSALV01SR x 100 units

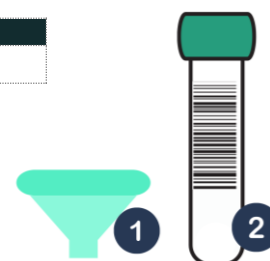

Components per unit of RBMSALV01SR:

COMPONENT	REFERENCE	PRESENTATION	STORAGE
Collection funnel	EMB	1 unit	Room Temperature
Saliva sample collection tube	TR5ML	1 unit	Room Temperature
Instructions for use	-	1 unit	Room Temperature

Material required but non provided:


- Preservative solution (available in other references)
- Reagents for subsequent nucleic acid extraction
- Reagents for subsequent nucleic acid amplification

STORAGE AND STABILITY

Kits should be stored at room temperature (10-30°C) until the expiry date stated on the label. Keep in a cool, dry place. Dispose of according to specific methods for the destruction of biological material. If this is not possible, contract an authorised disposal company.

SALIVA SAMPLE COLLECTION PROCEDURE

1. Do not eat, drink, smoke or chew gum for at least 30 minutes prior to sample collection. It may take several minutes to obtain **1 or 2 ml of saliva**.
2. Insert the **collection funnel (1)** into the **sample collection tube (2) ref. TR5ML**. Rub your cheeks with your tongue and teeth to stimulate saliva secretion.
3. Collect saliva **up to the level of the 1 or 2 ml line**. Remove the bubbles excess by tapping on a hard surface to collect the right amount.
4. Remove and discard the collection funnel.
5. Close tightly the **sample collection tube (2) ref.TR5ML** with its green cap.
6. The saliva sample is now ready for shipping and processing.

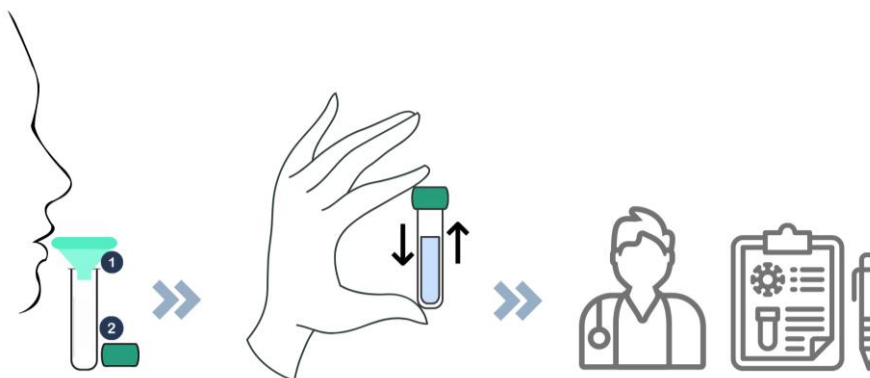

WARNINGS AND PRECAUTIONS

- Keep away from children.
- Wash your hands before and after sample collection.
- Do not Re-use. Do not use if the package is damaged.
- Do not use the test kit beyond expiration date.
- Keep away from heat and direct sunlight.
- Any serious incident involving the product should be reported to the manufacturer and the health authority.

PROBLEMS GUIDE AND SOLUTIONS

For any further questions or queries about the protocol, please contact DURVIZ's technical service (durviz@durviz.com)

SYMBOLS DEFINITION

REF	Catalogue number		Manufacturing date
	Read instructions for use		Keep dry
	Expiry date		Temperature limit
LOT	Lot		Do not use if the package is damaged
	Contains sufficient for n test		Single-use only
	Manufacturer		Caution
CE	CE marking	IVD	In vitro diagnostic medical device
UDI	Unique device identification		



Durviz S.L. Leonardo Da Vinci 10, 46980 Paterna, Valencia, España.
Tel. +34 96 136 61 07 E-mail: durviz@durviz.com - Internet: www.durviz.com