REAL MINI & MIDI SYSTEM

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1. PRODUCT DESCRIPTION

The Mini & Midi System is a device designed for the collection, preservation, transport, filtration and concentration of intestinal parasites present in faeces. The system offers a fast, clean and safe protocol, avoiding direct contact with the sample and reducing the risk of contamination or exposure.

It is available in two formats:

- **Mini System:** contains 4 ml of fixative and is compatible with centrifuge rotors and adapters for 15 ml conical tubes.
- **Midi System:** contains 10 ml of fixative and is compatible with centrifuge rotors and adapters for 50 ml conical tubes.

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The system can be supplied with different fixing media, depending on the user's needs:

- **SAF:** Acetic acid buffer with formaldehyde. A classic, economical medium with high fixing power, but toxic, requiring a fume hood and not compatible with molecular techniques.
- **ECOSAF:** Contains acetic acid buffer, methanol and a lower concentration of formaldehyde. Safer than SAF, but still requires an extractor hood and is not compatible with molecular techniques.
- **TOTAL FIX:** Mixture of alcohols and zinc sulphate. Recommended by Durviz for its safety, good fixing power and compatibility with molecular techniques.
- **No fixative:** Empty tubes for fresh samples. Useful for immediate observations, although they present a risk of degradation and require controlled handling.

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2. TECHNICAL SPECIFICATIONS

- The tube material is polypropylene, 100% recyclable.
- Fixative volume: 4 ml (Mini) /10 ml (Midi).
- Compatible with centrifuge rotors/adapters for 15 ml (Mini) and 50 ml (Midi) conical tubes.
- Contains 3 mm diameter glass beads for easy homogenisation.
- Leak-proof seals incorporated in both caps.
- Double horizontal filter of 400 and 250 μ m.

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3. STORAGE AND DISPOSAL

- Store at room temperature.
- Use before the expiry date indicated on the label. If the sample is preserved in the fixative, it will remain stable until that date.
- Dispose of according to specific methods for the destruction of material containing biological content.
- If it contains fixative, dispose of them through an authorised hazardous waste manager.

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4. PRECAUTIONS

- Wash your hands before and after handling the sample.
- Treat samples as potentially infectious.
- Avoid using anti-diarrhoeal medication or laxatives before collection.

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This system may contain a **chemical fixative** (SAF, ECOSAF or TOTAL FIX) **with toxic and irritant properties**. It must be handled in accordance with the specific instructions and precautions indicated for each fixative and the corresponding **Safety Data Sheet** (SDS) must be consulted before use.